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




2019

# XDRIP ANDROID

Xdrip configuration manual for Bubble in Android mobiles



# 1. XDRIP+ for Android

XDRIP+ is an Android compatible application available for download from the Internet and made by the developers community for its usage with Bluetooth devices form glucose monitoring.

As freeware this APP is not available for download from Google Play Store and it must be downloaded by the user in one of the URL where its has been released by its developers.

Download and software installation is responsibility of the user and neither its usage or any damage caused by the manipulation of the APP.




## 2. Software download

You can download the last version of the APP following this link::

**[https://bubbleshop.eu/download/xdrip\\_Bubble.apk](https://bubbleshop.eu/download/xdrip_Bubble.apk)**




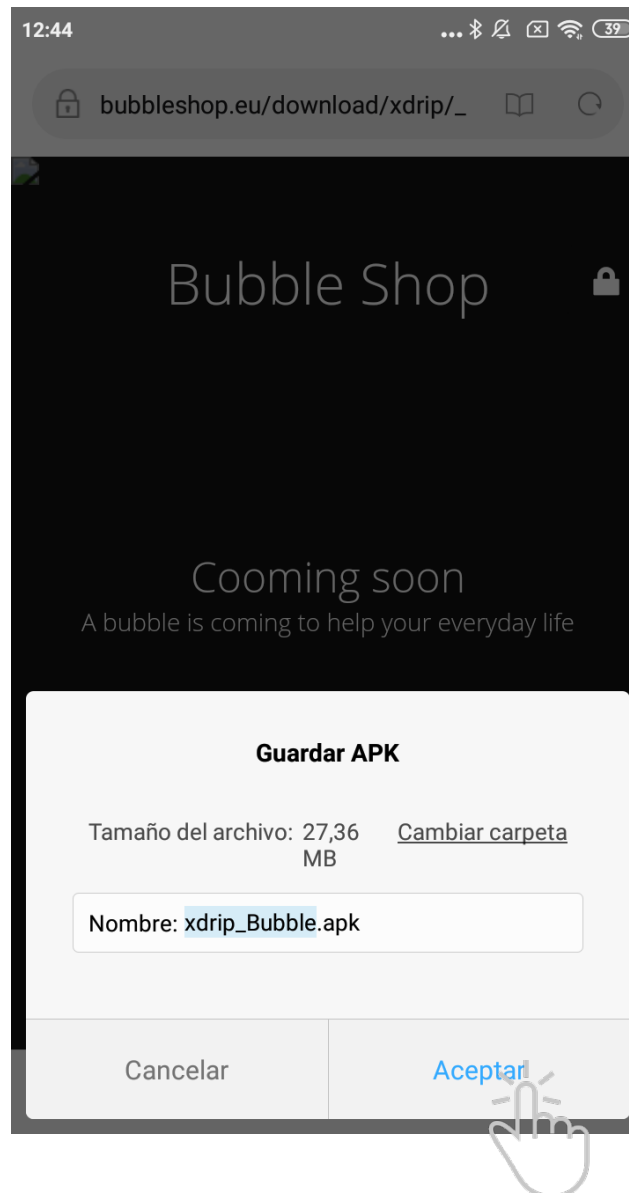


### ¡Important!



Bubble manufacturer does not have any relation with XDRIP and its developers.

Downloading this archive you assume all the responsibility for any damage or modification caused in your mobile phone.

Once downloaded. Install

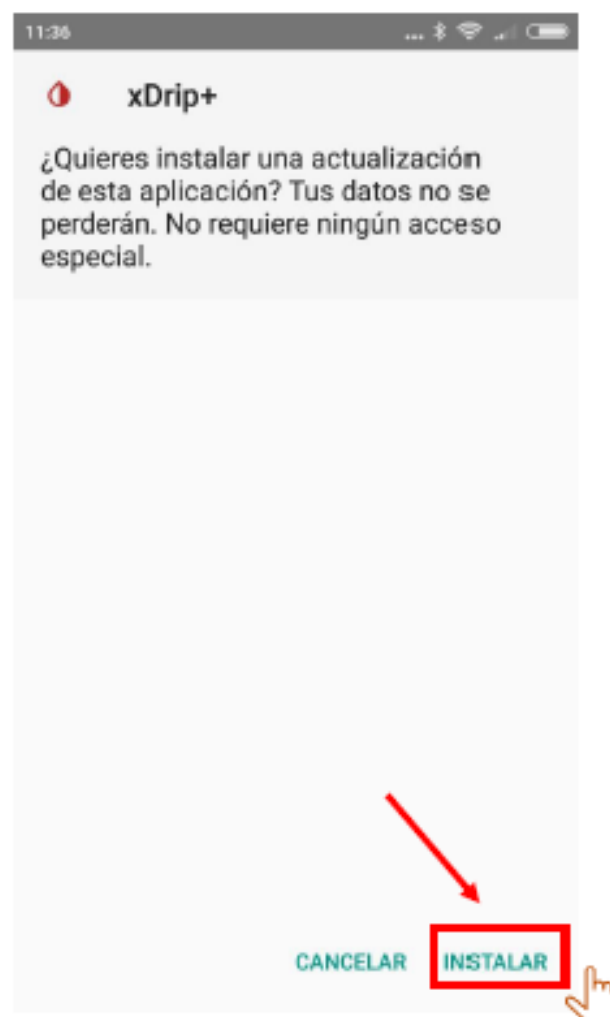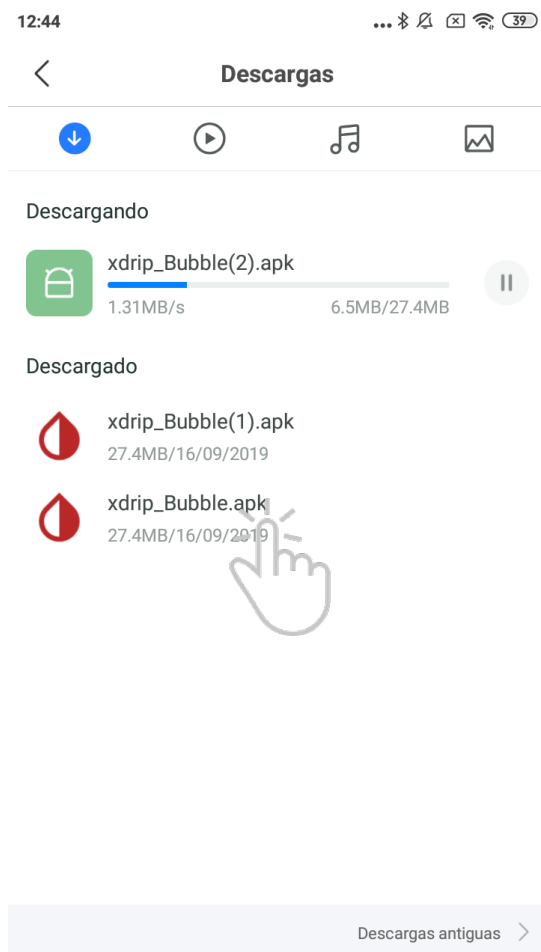

Open the app once installed

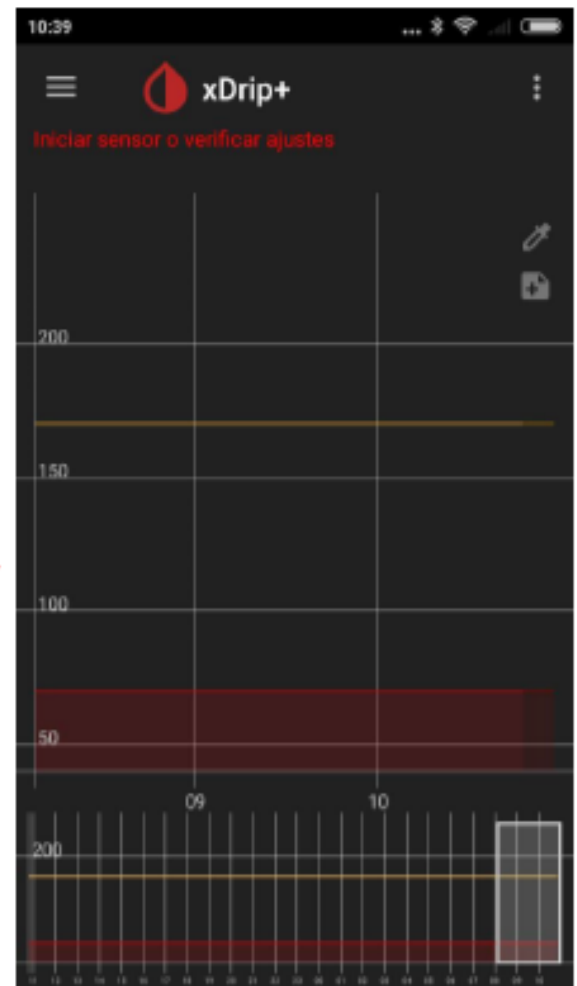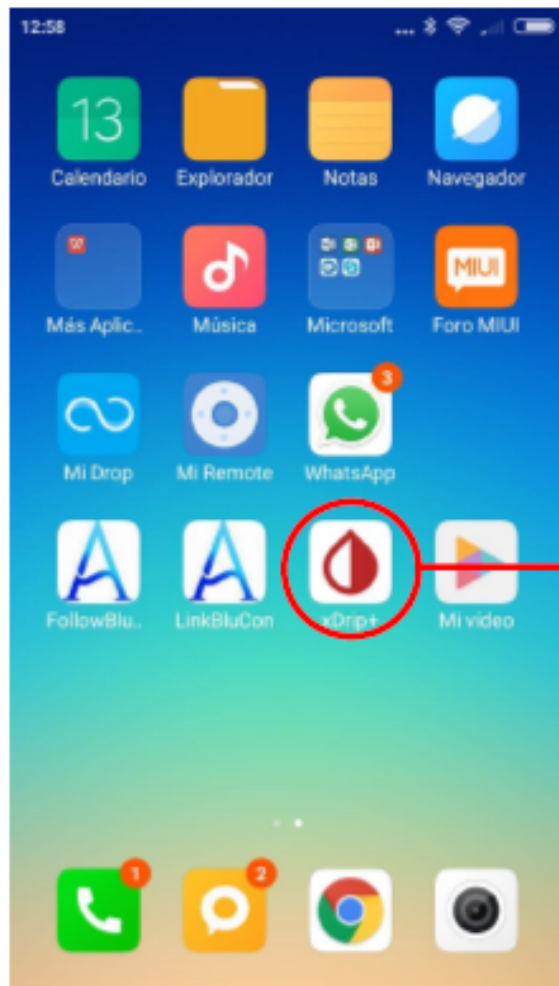

## 3. Pairing Bubble



**Bubble must always be paired through one of the APPs like XDRIP.**



**It is not possible to pair your device through the mobile phone Bluetooth menu.**

**If the application does not detect your Bubble unit please make sure that it is charged. If it keeps unconnected please make reset as indicated in the manual.**





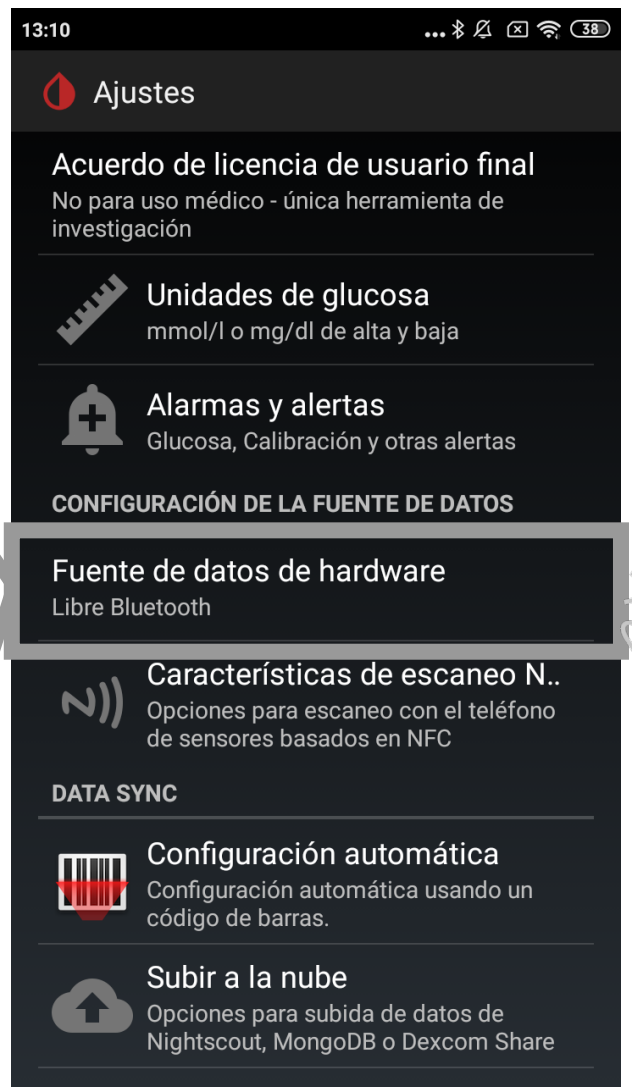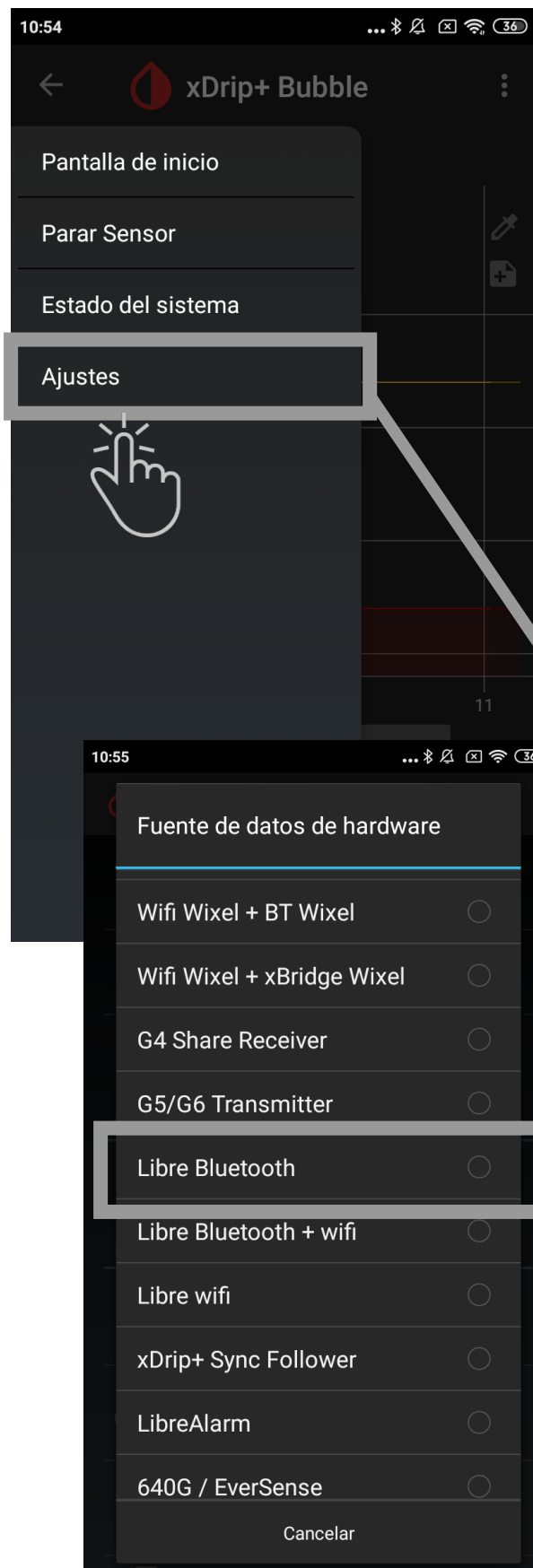

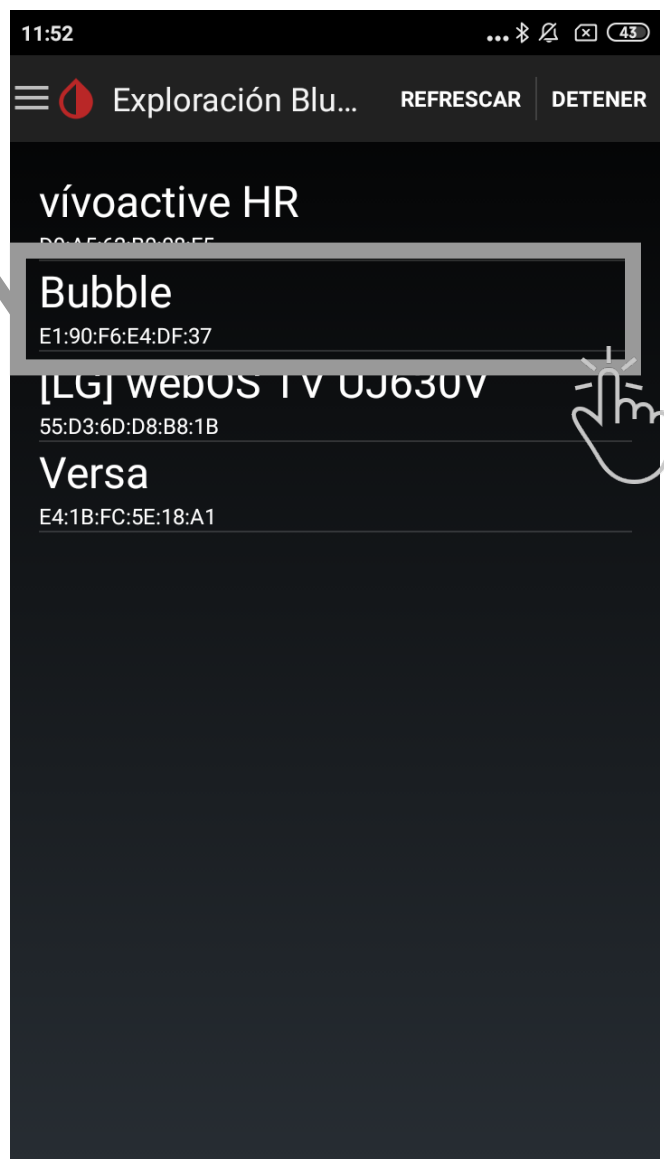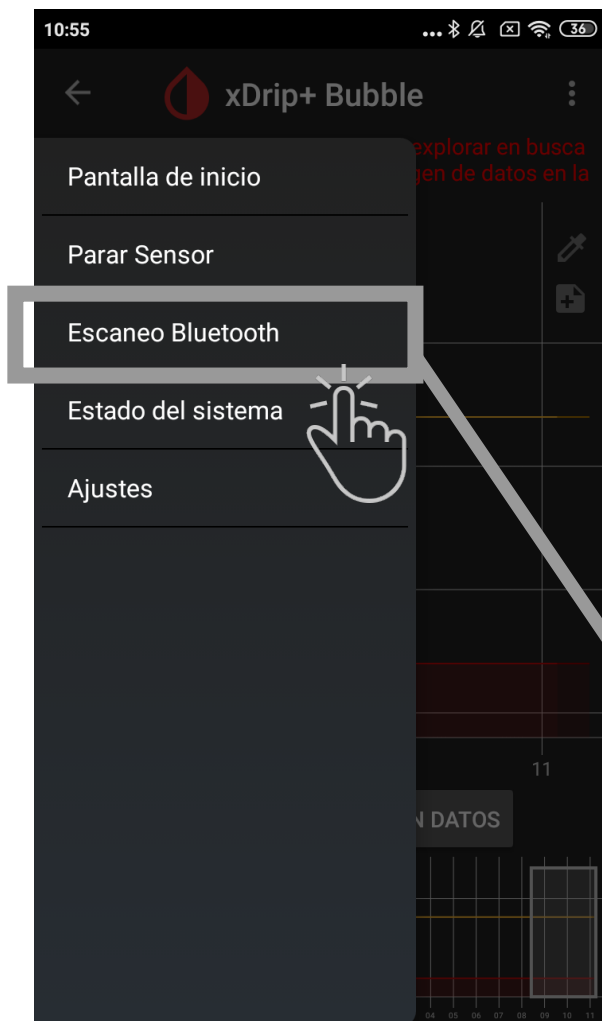


## 4. Initializing the sensor

Once you have Bubble paired, if you do not have readings you must initialize the sensor.

### IMPORTANT

Sensor must always be initialised with Abbott Freestyle Libre® reader or LibreLink® app.

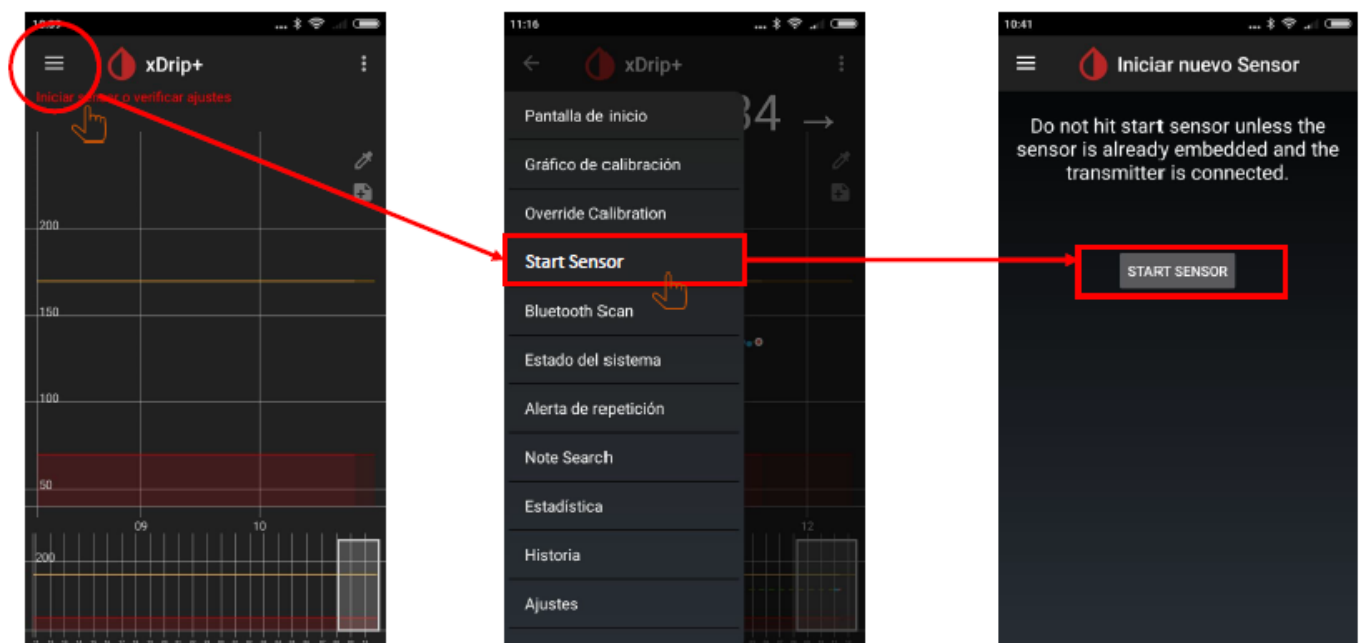





# BUBBLE

KEEPING YOUR DIABETES LEVELS UNDER CONTROL